TecnoLaser

Manufacturer since 1979

Ultrasounds device, with cavitation effect, sweep mode emission

It is the ultimate, non-invasive equipment, for the treatment of cellulite. Indeed the cavitation facilitates the reduction of fat cells volume mainly throughout the lymphatic and urinal systems.

The sweep mode emission optimizes the cavitation effect.

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VELA SOUND

CAVITATION AND ITS EFFECTS The cavitation is a physical phenomenon occurring into liquids in turbulence condition, consisting in micro-bubbles creation, filled with gas/vapour of the same liquid. This effect can also be obtained through high energy ultrasounds (u/s). Indeed the (u/s), crossing the human body, tend to move the molecules from each other and, if the energy is sufficient, the creation of these micro-bubbles will happen. These ones, imploding, give origin, in the same micro-volumes, to very high temperature, pressure and heat exchange in fractions of microseconds. The u/s, in the medical field, are mainly used to remove (through cavitation effect) renal calculus, for echography (low energy, without cavitation effect), for physiotherapy, etc. In our case, the effect of cavitation on the body gives origin to a higher permeability of fat cells whose contents (fat acids) will then be conveyed (to be eliminated) to the lymphatic and urinary systems.

CHARACTERISTICS OF THE DEVICE

Absorbed power	180 VA
Dimensions	40 x 40 x 145 (h) cm
Weight	16 kg
Peak power	25W adjustable from 10 to 100%
Specific power emitted	Max 2W/cm ²
Ultrasound frequency	25 kHz
Mode of emission	Modulated emission, with sweep mode
Emission head diameter	6,2 cm
Emission head effective area	30 cm ²
Emission time	Programmed up to 30 consecutive minutes

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